




Secuencias de fresado para Touareg CloseFit™

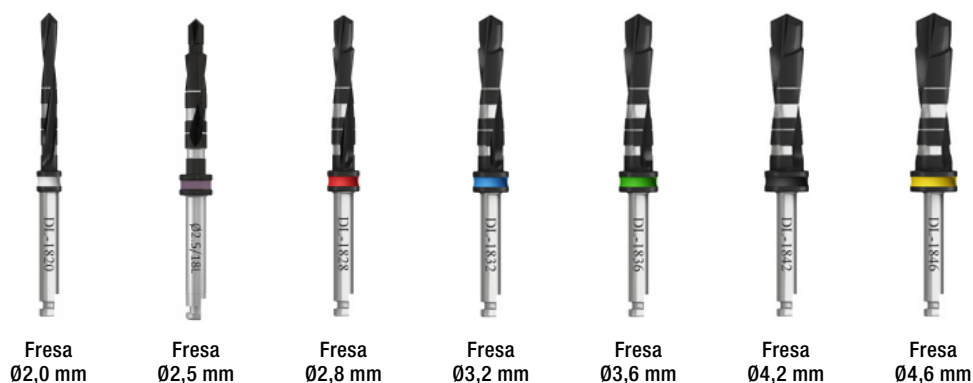

Fresado en huesos Tipo IV (blando)

UNP Ø 2,75 mm	•						
NP Ø 3,0 mm	•						
RP Ø 3,5 mm	•	—	•	•			
WP Ø 4,3 mm	•	—	•	•	•		
WP Ø 5,0 mm	•	—	•	•	•	•	

En huesos Tipo II y III (moderado y denso)

UNP Ø 2,75 mm	•	•					
NP Ø 3,0 mm	•	—	•				
RP Ø 3,5 mm	•	—	•	•			
WP Ø 4,3 mm	•	—	•	•	•		
WP Ø 5,0 mm	•	—	•	•	•	•	•

En huesos Tipo I (muy denso)

UNP Ø 2,75 mm	•	•					
NP Ø 3,0 mm	•	—	•				
RP Ø 3,5 mm	•	—	•	•			
WP Ø 4,3 mm	•	—	•	•	•		
WP Ø 5,0 mm	•	—	•	•	•	•	•

Nota: solo en zona cortical

Nota: en huesos muy densos podría requerir pasar más de una vez la fresa indicada o bien insinuar la punta de la siguiente fresa en la secuencia. Por su diseño de dos etapas, las fresas de Adin incorporan función avellanadora.

